


Merlin VGA

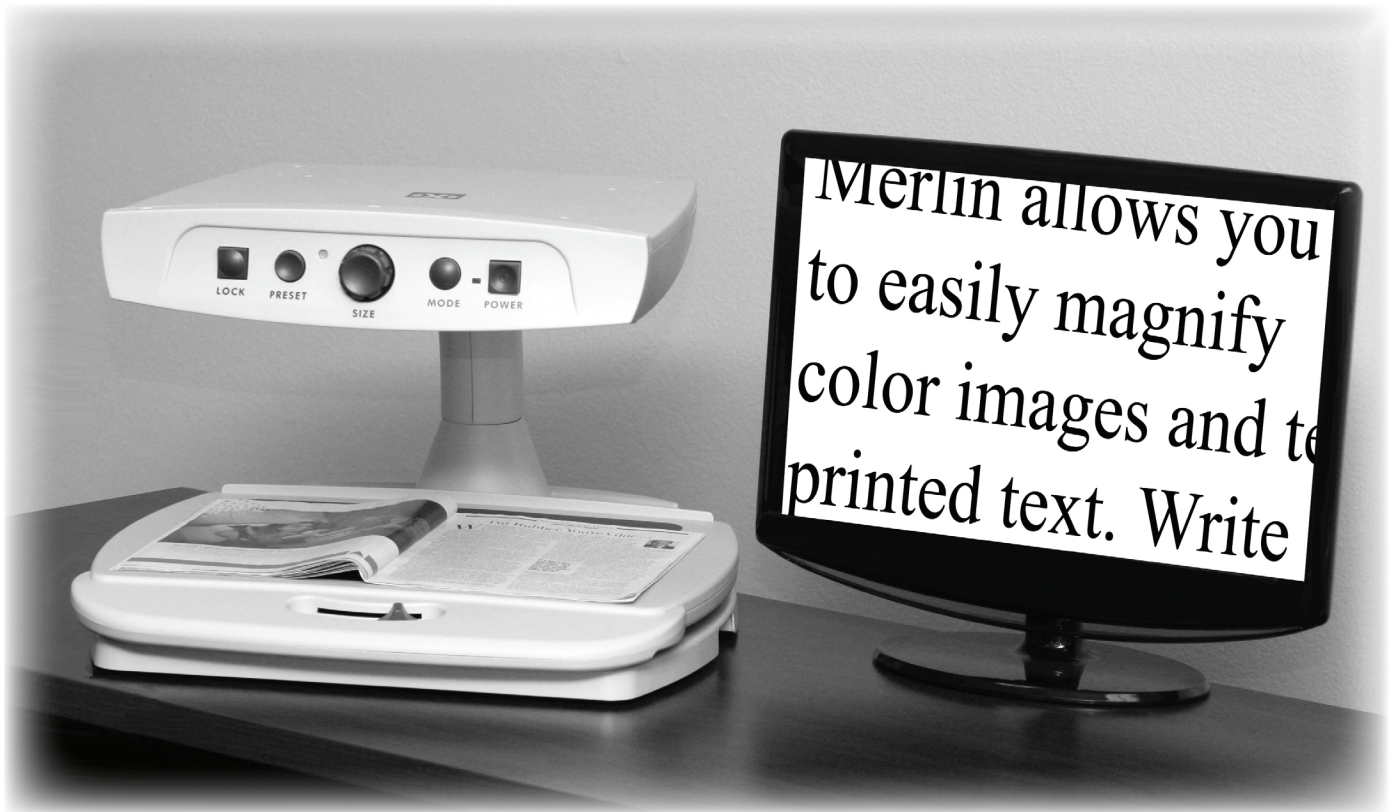

Users Manual

FCC Compliance Notification

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Unauthorized changes or modification to this system can void the user's authority to operate this equipment.

If your Low Vision device has an option to connect to a TV, it must be used with shielded interface cables in order to meet FCC emissions limits for Class B digital devices. It is the responsibility of the user to use the shielded interface cables supplied with the equipment.

About Your Merlin VGA

Congratulations on your purchase of the Merlin VGA from Enhanced Vision. The Merlin VGA is a full-color, auto-focus video magnifier system designed to enhance the lifestyle of the visually impaired. The Merlin VGA easily connects to any standard computer monitor equipped with a VGA input.

Merlin VGA enables individuals with low vision to read, write, view photos, enjoy crafts and hobbies, and more – in color, black-and-white, or enhanced high-contrast positive or negative viewing modes. Merlin VGA also includes 3 different preset color combinations for both text and background to provide improved viewing when reading. An additional 25 color combinations are available and can be programmed. With easy-to-use buttons that are also distinguishable by touch, you can enlarge or reduce the image in seconds, change viewing modes for easier visibility. The auto focus feature on Merlin VGA allows for optimum image quality for reading. If desired, the focus lock feature may be enabled for improved image quality when writing.

Please read this booklet and familiarize yourself with Merlin VGA features and operation before using this system. Your Merlin VGA is ready to use once properly connected to a standard computer monitor equipped with a VGA input. By reviewing Merlin VGA features and controls before using it and following the directions carefully, you'll be able to enjoy all the great benefits from your purchase.

Enhanced Vision products are designed to give you the highest quality image, convenient and easy-to-use features and years of reliable service. Thank you for choosing the Enhanced Vision Merlin VGA.

Table of Contents

Safety and Care	4-7
Merlin VGA at a Glance	8-9
Setting Up Merlin VGA	10
Turning on Merlin VGA	11
Changing Viewing Mode	12-13
Using Color Select	14-16
Changing Magnification	17
Intelligent Focus Lock	18
Using the X-Y Table	19
Tips for Reading and Writing	20
Programming Set-up Options	21-22
Using the Object Locator	23
Programming Reference	24-25
Troubleshooting	26-27
Specifications	28-29
Other Products from Enhanced Vision	30-31
Contact Information	32
Warranty Statement	33-34

Safety and Care

To obtain the most benefit and life from your Merlin VGA, please read these safety precautions very carefully.

Merlin VGA is designed to enlarge and enhance images in order to offer improvement in the quality of life for people with visual impairment. The extent to which Merlin VGA can improve the ability to see images depends on a person's individual condition and level of visual impairment.

Merlin VGA is not designed to restore vision, correct eye conditions or halt degenerative vision loss. An eyecare professional can determine the usefulness and suitability of Merlin VGA for a patient's individual condition.

Precautions for Use

- Follow all warnings, cautions, and instructions marked on the product.
- Use the Merlin VGA on a stable, level surface such as a solid desk or table, according to its intended design specifications.
- **Enhanced Vision does not recommend placing an LCD or plasma monitor on top of your Merlin VGA!** The lightweight design of many newer LCD monitors (flat screen style) may cause them to tip over, causing serious damage to the unit or bodily injury to the user. For this reason, it is recommended to place your external monitor to either side of the Merlin VGA during normal use.
- Use the Merlin VGA indoors only – it should NOT be exposed to extreme or rapid changes in temperature or humidity. Do not place the Merlin VGA near a radiator, heater or areas exposed to long hours of direct sunlight. These conditions may cause damage to the unit's electronics or optics.
- Do not use the Merlin VGA near flammable materials or liquids.

Additional Electrical Safety Precautions

- The power supply cord is a disconnect device for electrical equipment. The socket-outlet should be installed near the equipment and be easily accessible.
- This product is equipped with a 3-wire grounding type plug as a safety feature. This plug has a third (grounding) pin and can only be plugged into a grounding-type power outlet. If necessary, contact an electrician to replace your power outlet with a grounding-type power outlet.
- Do not use the Merlin VGA in, near or around water. The Merlin VGA is not designed to offer protection against contact with liquids of any kind.
- Use only the power supply provided with your Merlin VGA. Using unauthorized power supplies may cause serious damage to Merlin VGA's electronic components.
- Prevent the risk of electrical shock by NOT attempting to disassemble the unit. Please refer all service to an authorized repair facility.
- To avoid personal injury or damage to the Merlin VGA, do not make any other type of connections to the power input jack other than the included power supply.

Safety and Care

Additional Mechanical Safety Precautions

The Merlin VGA contains moving parts. Please use caution when moving the X-Y table or when transporting the Merlin VGA to avoid possible injury to hands and fingers.



! CAUTION
MOVING PARTS!

To Move Merlin VGA to Another Location

- If you are repositioning Merlin VGA a short distance across a desk or table, make sure you do not pull on any power cords or connector cables.
- Disconnect and secure all power and VGA cables and lock the X-Y table before moving the unit.
- Carry Merlin VGA only by the two recessed handles located on either side of its base. **Do not carry it by the LCD monitor arm.**

Caring for Merlin VGA

- Always turn the power OFF for the Merlin VGA and the monitor when the system is not in use.
- At least once per week, clean the Merlin VGA with a soft dry cloth or an electrostatic dusting wipe. DO NOT use liquid cleaners on its plastic surfaces. No additional methods of sterilization or cleaning are recommended or required.

Susceptibility to Interference:

There may be temporary degradation of the image when the Merlin VGA is subjected to a strong radio frequency field, electrostatic discharge or transient electrical noise. An electrostatic discharge (caused by static electricity) may cause the screen to go black. If this happens, switch the unit off and on again.

Compliance Warnings:

- Electromagnetic compatibility: The use of accessories and cables other than those specified by Enhanced Vision may result in increased emissions or decreased immunity.
- Placement of equipment: The Merlin VGA should not be used adjacent to or stacked on top of other equipment.





Merlin VGA at a Glance

Take a few moments to familiarize yourself with the components of your Merlin VGA (refer to next page):

1. Merlin VGA
2. Power button
3. Viewing Mode button
4. Size (Magnification) dial
5. Preset/Object Locator button
6. Focus Lock/Auto Focus Button
7. Light ON/OFF Switch (*located underneath unit*)
8. X-Y table
9. X-Y table locking lever
10. Recessed Handles for Carrying Merlin
11. VGA Output Cable*
12. Power Input
13. Power Supply (*style may vary depending upon country*)
14. Power Cord (*style may vary depending upon country*)

**Note: Your Merlin VGA must be connected to a standard computer monitor or LCD/Plasma display equipped with standard VGA inputs (not supplied)*

Merlin VGA at a Glance

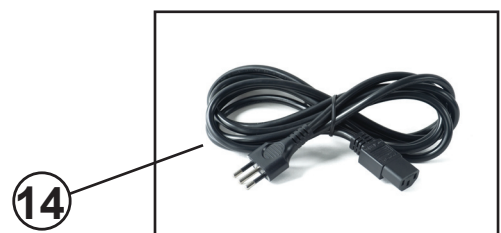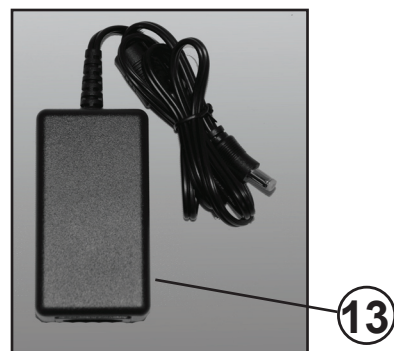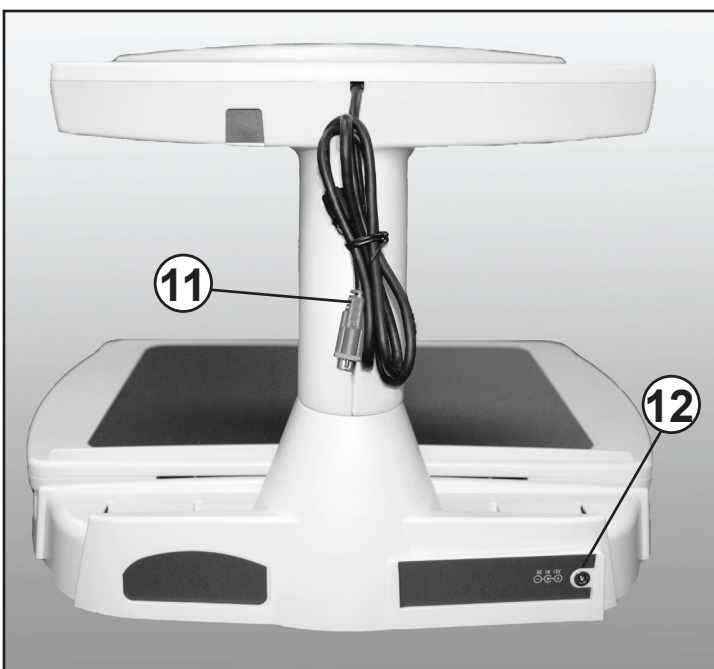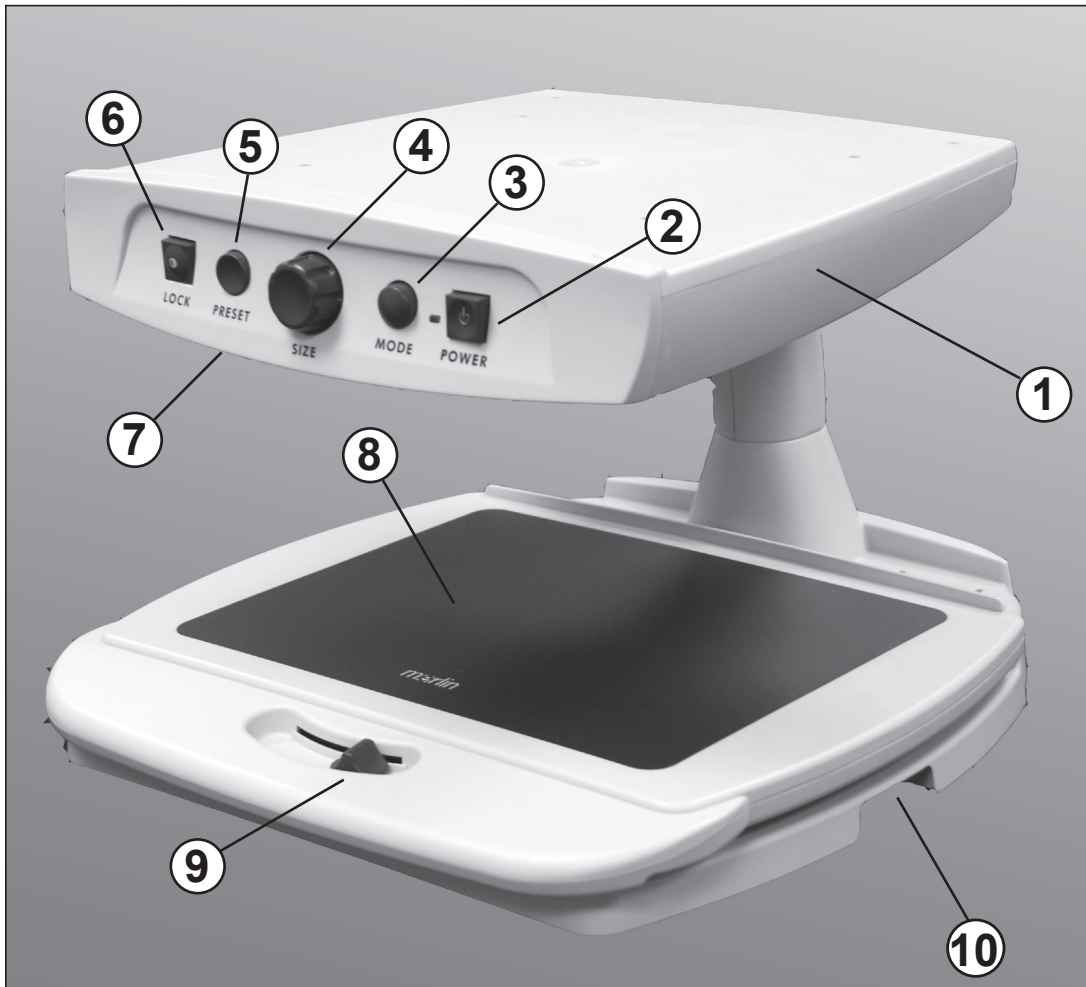

Setting up Merlin VGA



Figure 1

1. Find a steady and level location, such as a table or desk, where you will be comfortable using Merlin VGA. Carry the unit only by the two recessed handles on the side of the base for a secure grip (see Figure 1). **Do not lift the unit by the top!** Place the Merlin VGA and your VGA monitor on the table or desk at a comfortable height.

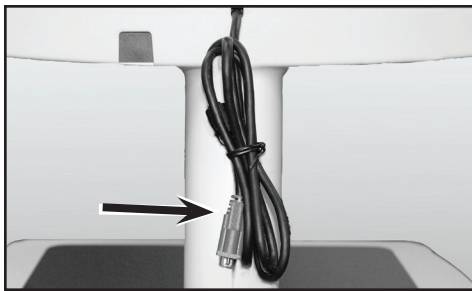


Figure 2

2. Connect the VGA output cable of the Merlin VGA to the VGA input on your external VGA monitor (see Figure 1). Connect the AC power plug of your monitor to an available wall outlet.

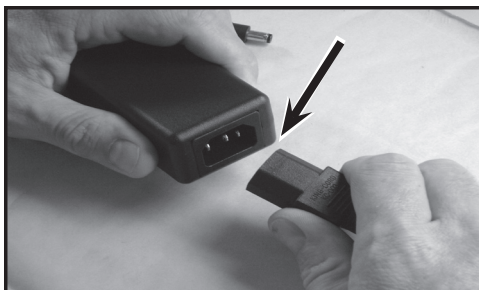


Figure 3

3. Connect the end of the supplied power cord to the Merlin Power Supply transformer (see Figure 3 and items #13 & #14 on page 9).


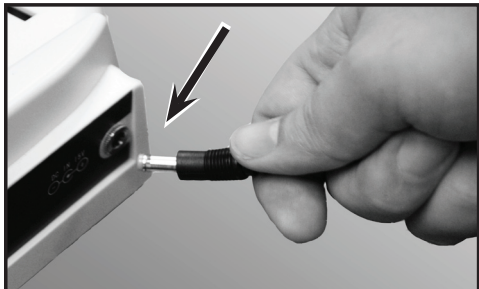


Figure 4

4. Connect the power lead from the transformer to the Merlin VGA Power input on the rear of the unit as shown (see Figure 4). Finally, connect the AC power plug to an available wall outlet.

Turning On Merlin VGA

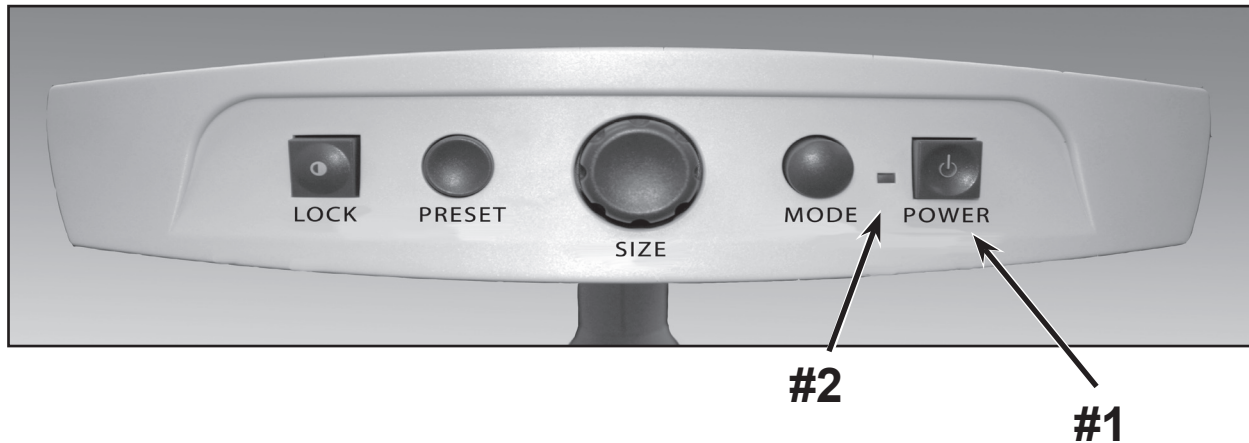

Merlin VGA Power Button


The Power button on your Merlin VGA is located on the far right of the control panel (item #1 above).

- Press the Power button. Notice the green light will illuminate on the Control Panel, indicating that the power is "ON" (item #2 above).
- Once properly connected as described on the previous page, an image should appear on your VGA monitor within a few seconds.

If the green power indicator does not illuminate or if an image does not appear on your LCD monitor, please check the power supply connections (review steps 2-4 on page 10 for details).

Note: If your Merlin VGA is plugged into a power strip or surge protector (recommended), make sure its power switch is turned on before attempting to turn on Merlin VGA. If the power strip/surge protector comes with a circuit breaker, make sure the circuit breaker has been reset before attempting to turn on Merlin VGA.

Changing Viewing Modes

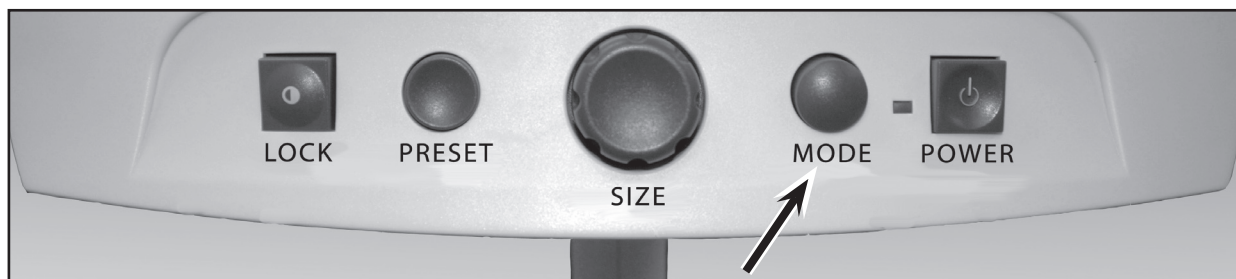

You can choose a viewing option by pressing the MODE button on the control panel. Each press will cycle through all 7 viewing options.

The available viewing options are:

- Full Color Image
- Black and White Image
- Enhanced Positive Image
- Enhanced Negative Image
- Color Select 1
- Color Select 2
- Color Select 3

**Crisp, Clear
Text Images!**

Enhanced Positive

**Crisp, Clear
Text Images!**

Enhanced Negative

Full Color Image

This mode is used most often because it gives you a true color representation of the image that is being magnified. When Merlin VGA is first turned on, it will display the image in full color.

Black and White Image

Based on your preference, you can choose the black-and-white mode.

Enhanced Positive Image

This mode provides a white background with black text. This mode increases the contrast to offer crisp text images for reading.

Enhanced Negative Image

This mode provides a black background with white text. This mode increases the contrast to offer crisp text images for reading.

Color Select 1

This mode provides a blue background with yellow text. This is one of the most commonly used color combinations.

Same as Color Mode combination #28 (see page 16)

Color Select 2

This mode provides a yellow background with black text.

Same as Color Mode combination #16 (see page 16)

Color Select 3

This mode provides a black background with yellow text.

Same as Color Mode combination #2 (see page 16)

Using Color Select

Merlin VGA was designed to provide optimum viewing for people with certain visual conditions by offering an expanded range of choices in both text and background colors.

This combination of text and background colors comprises a color select mode. The Merlin VGA offers the following:

- 28 text/background color combinations (see page 16)
- 3 factory preset color select modes (see page 13)
- 2 additional user color select modes

Both the preset and user color select modes can be independently programmed to your preferences, using the procedure listed below.

How to Program a Color Select Mode



With Merlin VGA turned ON, place a piece of reading material on the X-Y table so that it appears on your monitor at a magnification size that is comfortable for viewing. Follow these steps to program any of the 5 color select modes:

Step 1: Press and hold the MODE button (approximately 5 seconds) until the words “ENTERING SETUP” appear on your LCD monitor.

ENTERING SETUP

Step 1

Using Color Select

Step 2: Rotate the SIZE dial to scroll through the available menu options. Choose COLOR SELECT (see the photo at right) and press the MODE button again.



COLOR SELECT

Step 2

Step 3: Rotate the SIZE dial to choose the Color Select number (1-5) you wish to program. Press the MODE button and choose one of the following options:

- a) Rotate the SIZE dial to cycle through the color combination of your choice (1-14). To access color combinations 15-28, press the MODE button first, before rotating the SIZE dial again. (Please refer to the list on next page)
- b) OR, if you want to turn OFF the current COLOR SELECT (1-5) option, select OFF by turning the SIZE dial clockwise. If no change is desired, select PREVIOUS MENU by turning the SIZE dial counter-clockwise.



OFF



PREVIOUS MENU

Step 3b & Step 5

Step 4: Press and hold the MODE button (~ 5 seconds) to register the color combination you've programmed into the Color Select.

Step 5: When you're finished, rotate the SIZE dial all the way counter-clockwise to PREVIOUS MENU and press the MODE button. Finally, rotate the SIZE dial all the way counter-clockwise to EXIT and press the MODE button to finish programming.



EXIT

Step 5

Using Color Select

Using the procedure outlined on pages 14-15, the following 28 color combinations can be programmed into any of the 5 color select programs, based on your personal preferences.

Color Mode:	Background Color:	Text Color:	Notes:
#1	Black	Green	
#2	Black	Yellow	*Default Color Select 3
#3	Black	Red	
#4	Black	Blue	
#5	Black	Purple	
#6	Black	Orange	
#7	White	Green	
#8	White	Red	
#9	White	Purple	
#10	White	Blue	
#11	White	Orange	
#12	Yellow	Red	
#13	Green	Yellow	
#14	Yellow	Blue	
#15	Green	Black	
#16	Yellow	Black	*Default Color Select 2
#17	Red	Black	
#18	Blue	Black	
#19	Purple	Black	
#20	Orange	Black	
#21	Green	White	
#22	Red	White	
#23	Purple	White	
#24	Blue	White	
#25	Orange	White	
#26	Red	Yellow	
#27	Yellow	Green	
#28	Blue	Yellow	*Default Color Select 1

Changing Magnification

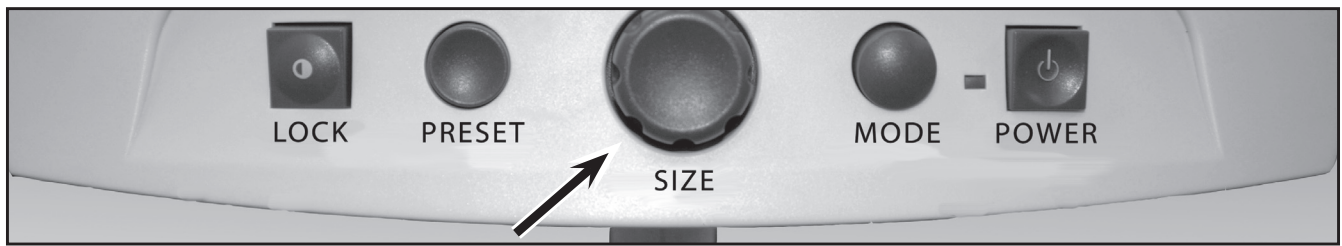


Figure 1

You can increase or decrease the size (magnification) of the image by using the SIZE dial located at the center of the Merlin VGA control panel (see Figure 1).

There are sixteen levels of magnification available on the Merlin VGA.

- To increase the size of the image, turn the dial clockwise, or to the right.
- To decrease the size of the image, turn the dial counterclockwise, or to the left.

The size you've selected will be displayed in the upper left corner of your monitor (see Figure 2).



Figure 2

Intelligent Focus Lock



When Merlin VGA is first turned on, it is automatically set on **Auto Focus**. This is the most convenient setting for everyday activities, such as reading anything from a single page to a thick book, reading a prescription bottle, or working with arts and crafts. Alternately, **Focus Lock** allows you to maintain focus on a specific object at a fixed distance, such as writing. Merlin VGA's Intelligent Focus Lock feature will automatically refocus on the object you are viewing after changing magnification size.

To Use Focus Lock for Writing

1. Place your note paper, stationery, checkbook or other writing material on the X-Y table.

2. Press the [LOCK] button on the control panel. The letters “FL” will appear and remain in the upper left corner of the monitor. In this mode, the Merlin VGA will now keep your writing in focus, rather than your hand when it comes into the camera’s frame.

If you change magnification sizes, Intelligent Focus Lock will automatically refocus on the desired object or page.

3. To return to auto focus mode, press the [LOCK] button again. The letters “AF” will appear in the upper left corner of the monitor.

FL

AF

Locking the X-Y Table

To lock the X-Y table, slide the lever all the way to the right until it clicks and locks (A). Unlock the X-Y table by sliding the lever to the right again and release the lever (B).



Reading With the X-Y Table

1. With the X-Y table unlocked, place a book, newspaper, etc., on the table.
2. Use the magnification dial to enlarge the image to a comfortable viewing size.
3. Glide the X-Y table to the left, which will make the words flow across the screen for reading.
4. When you have reached the end of a line, glide the table back to the beginning of the line.
5. Move the table away from you slightly to display the start of the next line.
6. Repeat steps 4, 5 and 6 to continue reading.

Tips for Reading and Writing


Reading with Merlin VGA

- New users of video magnifier systems have a tendency to over-magnify their reading material. Make the type large enough to see it comfortably, but not so large that it becomes tedious to scan across a page.
- Use the X-Y table to move reading materials under the camera, rather than moving the material manually, except small items such as a checkbook or a business card.
- New users should use two hands when moving the X-Y table for reading, until they become comfortable using one hand.

Writing with Merlin VGA

- Beginners should start at smaller magnification sizes, and then increase magnification as they become more comfortable using Merlin VGA for writing.
- The smaller the magnification size the bigger you tend to write; the higher the magnification size the smaller you'll write.
- Use a pen that makes a bold line, such as a heavy point pen, a felt-tip marker or a fine point marker, for easier visibility.
- It might be helpful to use lined paper to more easily find and keep your place.
- It might be easier to keep your place if you write a few words at a time, then move the X-Y table and write again, rather than moving continually as you write.
- Write in either color, black-and-white or positive viewing modes.
- Depending on ambient lighting conditions within the room, it may be helpful to turn the built-in light On/Off (see item #7 on page 8).

Programming Setup Options

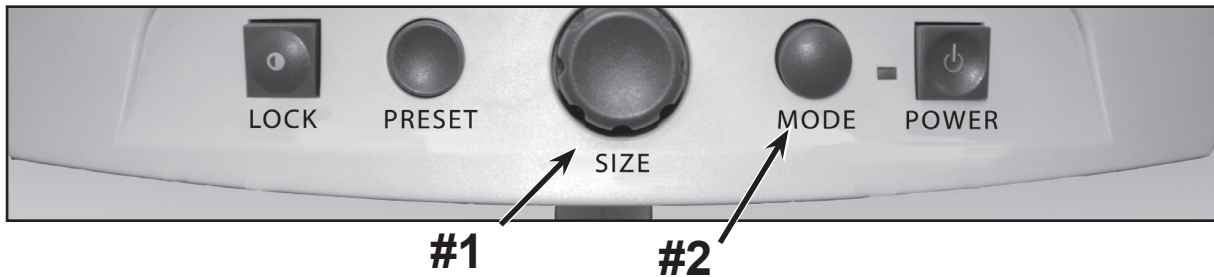


The Merlin VGA can be programmed in its set-up mode. This allows you to customize your experience when using the Merlin VGA. There are 5 options you can select within the programming set-up mode:

- Exit
- Screen Messages (On/Off)
- Black & White Mode (On/Off).
- Color Select (On/Off)
- Color Select (28 programmable color combinations, see page 16)

Entering Programming Setup:

1. Rotate the SIZE dial (#1 above) all the way counter-clockwise, to the lowest size. This will ensure that you start at the beginning of the menu options when you enter programming set-up.
2. Press and hold the MODE button (#2) for approximately 5 seconds or until "ENTERING SET-UP" is shown on the display. Momentarily, the screen will fade to black and the word "EXIT" will appear on screen.
3. Rotate the SIZE dial (#1) to the right to select each of the five different set-up options.
4. Press the MODE button (#2) to activate or deactivate each programming option.
5. Rotate the SIZE dial (#1) to make changes to the desired option.

Programming Setup Options

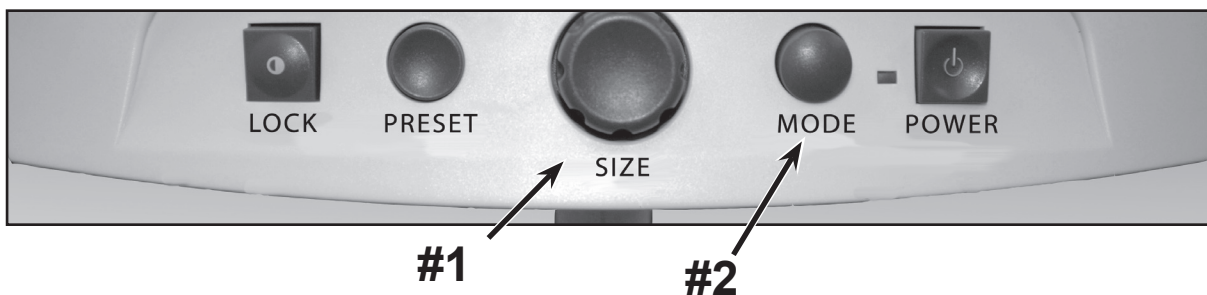

Exit: After making changes to the programming options listed below:

- 1) Rotate the SIZE dial (#1 above) all the way counter-clockwise until “EXIT” appears on the display.
- 2) Press the MODE button (#2 above) to exit and return to normal operation.

EXIT

Screen Messages (On/Off): If Screen Messages are set to OFF, the Merlin VGA will not display magnification sizes or viewing modes. From factory default, Screen Messages are set to “ON”

Black & White Mode (On/Off): If Black & White Mode is set to OFF, the Merlin VGA will skip the availability of the Black & White Color Modes. From factory default, Black & White Mode is set to “ON”

Color Select (On/Off): If Color Select is set to OFF, the Merlin VGA will skip the availability of the Color Select Modes. From factory default, Color Select Mode is set to “ON”

Color Select: If this option is selected, up to 5 different color select modes can be programmed, from a choice of 28 different color combinations (see pages 14-16)

RSVD: In the interest of continuous product improvement, these options are reserved by the manufacturer for future programming and cannot be modified at this time.


Using the Object Locator



Merlin VGA's object Locator feature can help you quickly locate something you're looking for on a page, or a specific part of an object you want to view more closely.

To locate the desired object:

1. Press and hold the [PRESET] button. Merlin will automatically zoom out to the widest field of view, and a locator "target" (as shown at right) will appear in the center of your display screen. Continue to hold the button down while performing the next two operations.

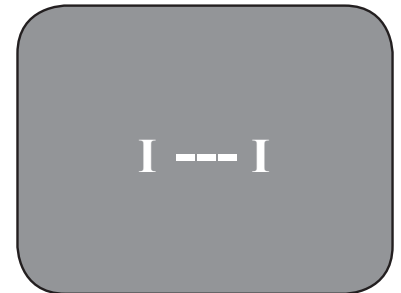

Note: Holding the [PRESET] button down for more than 10 seconds will cause the locator target to disappear.

2. Use the X-Y table and move your work or reading material until you have located the desired item within the "target" area.

3. Release the [PRESET] button and the Merlin VGA will magnify the item you've located within the targeted area.

Programming Reference

The Merlin VGA has advanced capabilities for modifying the default language used within the setup menus. These settings are completely optional and should not be changed under normal circumstances. The chart listed below is for reference purposes only. Please see your Enhanced Vision representative or call our toll-free technical support should it be necessary for you to modify these settings.

Merlin VGA Programming Reference

MERLIN 2 CLASSIC & PLUS FEATURES MENU

Secret 4321 code to enter
menu:


Size 4, then Hit Mode
Size 3, then Hit Mode
Size 2, then Hit Mode
Size 1, then Hit Mode

1st MENU LEVEL

FEATURES MENU

Displayed for 2 seconds
then goes to level 2 menu

2nd MENU LEVEL
LANGUAGES

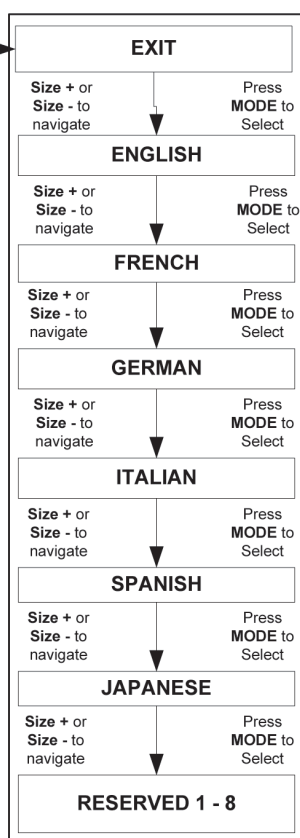

Merlin VGA Programming Reference


MERLIN 2 CLASSIC & PLUS USER MENU

Press MODE for 5 seconds to enter MENU


1st MENU LEVEL

ENTERING SETUP
Displayed for 2 seconds then goes to level 2 menu


2nd MENU LEVEL

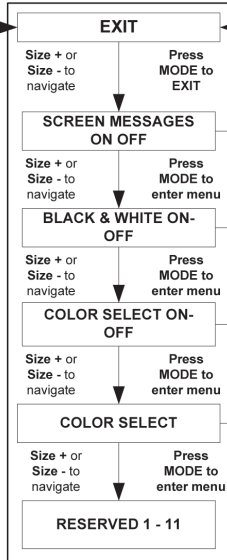


SCREEN MESSAGES SUB MENU

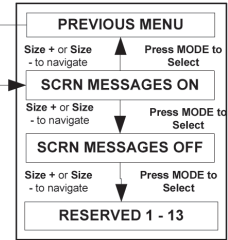


BLACK & WHITE SUB MENU

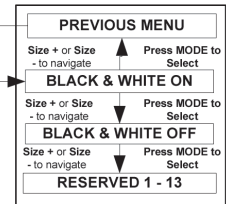


COLOR SELECT SUB MENU

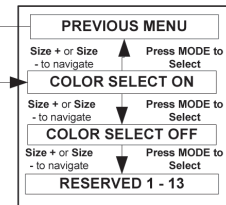

COLOR SELECT SUB MENU

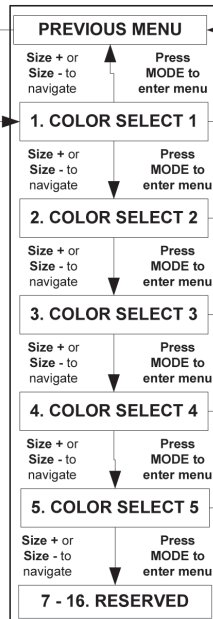

COLOR SELECT SUB SUB MENU

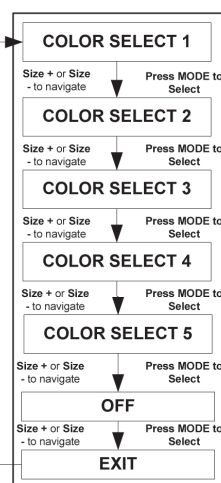

Troubleshooting

1. No Power to Merlin VGA

- Press the Power button on the Merlin VGA's. A light next to the Power button should turn illuminate, indicating the power is on.
- Make sure the power supply cord for the Merlin VGA is completely plugged into the AC outlet.
- Make sure the power supply cord is completely plugged into the power input jack on the lower bottom of the Merlin VGA.
- Check the power strip/surge protector's on/off switch and circuit breaker if the Merlin VGA is plugged into one.
- Check to make sure your VGA monitor is properly connected (both VGA signal and power) as detailed on page 10.

2. Blurry or Out-of-Focus Image

- Clean the protective glass covering the camera with the non-abrasive lens cleaning cloth provided with your Merlin VGA.
- Clean your monitor screen with a non-abrasive cloth sprayed with a glass cleaning solution.
- Turn the Merlin VGA light switch ON or OFF as needed to compensate for any glare or shadows caused by the ambient room lighting. The switch is located immediately underneath your Merlin VGA on the left hand side (*see item #7, pages 8/9*).
- Try changing the default focus setting from Auto Focus mode to Intelligent Focus Lock mode (*see page 18*). Depending on your specific application, either mode may help to improve focus.

3. Merlin VGA Lights are Not "ON"

- Check that Merlin has power. If not, see item 1, above
- Check that the light switch (*item #7, pages 8/9*) under Merlin LCD's control panel is switched to the ON position.

X-Y Table Does Not Move Smoothly

- Make sure the locking lever is positioned completely to the left.

X-Y Table Does Not Stay in Locked Position

- Make sure the locking lever is all the way to the right and that the handle pops up into the locking slot.

Replacement LED lights

The lighting for your Merlin VGA is provided by two 100,000-hour LED lights, designed to provide years of trouble-free service.

However, in the unlikely event that the LED lights need replacement, please contact your place of purchase or Enhanced Vision at (800) 440-9476 for replacement information.

If you have any questions or require technical assistance please call (800)440-9476, Monday – Friday from 8 a.m. to 5 p.m. PST.

Specifications

General (Dimensions & Weight):

Dimensions (base): 14.5"(H) x 19"(W) x 20.5"(D) (36cm x 48cm x 52cm)

Weight: 23lbs. (10.4kg)

Operating temperature range: 50-104°F / 10-40°C

Storage temperature range: 32-122°F / 0-50°C

Controls: Light Switch On/Off, Auto Focus/Intelligent Focus Lock, Preset, Magnification, Mode, Power On/Off

Magnification (based upon usage of a 19" VGA monitor):

2.7x (minimum) to 62x (maximum)

Power Supply:

110V-240VAC 50-60Hz

15VDC 1.2A

Equipment Classification (IEC 601-1):

Type of Protection: Class 1

Degree of Protection: Type B

Mode of Operation: Continuous

Color Modes:

- 4 Default Modes (True Color, Black & White, Enhanced Positive, Enhanced Negative)
- 5 Color Select Modes
 - 3 Default Modes: (Yellow on Blue, Black on Yellow, Yellow on Black)
 - 2 Programmable* Modes

*Up to 28 Programmable Color Select Mode Combinations are available

XY Table:

XY table size: 16" (406 mm) x 16 1/4" (413 mm), 3/8" retaining edge
Working space height: 7 3/8" (18.7mm)
Table height clearance: 1 3/8" (35mm)

Video output Resolution on VGA output:

VESA (DMT 1.0) SXGA (1280*1024) at 60Hz


Accessories:

Power Supply Transformer & Power Cord
Owner's Manual
Warranty Card

Limited Warranty:

3 Years Parts and Labor

Other Products from Enhanced Vision

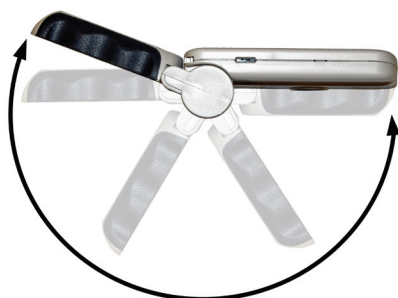

Pebble – Compact, Ultra-Portable Magnification!

The Pebble is a handheld electronic video magnifier that you can take anywhere. Weighing less than 8 ounces, Pebble features a uniquely-designed folding handle and includes both near-viewing and table reading modes. Small enough to fit in your pocket or purse, the Pebble is the ideal companion to provide magnification, on the go. Featuring a built-in 3 ½" LCD screen, the Pebble's Freeze-Frame feature allows you to capture any still image and adjust magnification. Six different viewing modes allow you to choose alternate color combinations based on your personal preference. Pebble can be powered using standard rechargeable batteries (included)

Magnification: 2X – 10X



Jordy – Two Low Vision Systems in One

For distance, intermediate and near viewing.

Magnification: 1X – 30X

Four viewing modes for optimum visibility:
Color, Black/White, Enhanced Positive,
Enhanced Negative



Jordy Docking Stand (optional)

Magnification: 4X – 40X



Other Products from Enhanced Vision

Acrobat LCD



With a built-in 19" LCD, the Acrobat is a 3-in-1 video magnifier ideal for near, distance and self viewing. By rotating the camera, you can read, write, groom, and magnify images at any distance. The Acrobat LCD offers a complete system for those at work, school or home.

Magnification: 2.5X – 65X (19" LCD)



Flipper – The Ultimate in Flexibility



Read, write, get more out of life! Our innovative camera design lets you magnify in almost any position with images in full color.

Magnification: 1X – 44X*

*With optional stand

Available with Color and Black & White Display Modes



Amigo



Amigo is a truly portable desktop magnifier. Optimal viewing is achieved with a tilting 6.5 inch viewing screen. Amigo provides 3.5 to 14x digital magnification and the lightest weight in its class. Amigo is battery operated and can be taken anywhere.



Contact Information

Enhanced Vision manufactures many different types of products tailored to meet the needs of our low vision customers. If you have questions about any of our products, please don't hesitate to give us a call.

Contact information:

(US Office, toll free): (800) 440-9476

(US Office) Phone: (714) 465-3400

Fax: (714)374-1821

(UK Office) Phone: +44 (0)115 9442317

Fax: +44 (0)115 9440720

www.enhancedvision.com

evsupport@enhancedvision.com

Warranty Statement

Enhanced Vision Merlin VGA Video Magnifier Warranty Policy

Enhanced Vision products are designed to give you the highest quality image, convenient features, ease of use and reliable service. **Merlin VGA has been inspected and tested prior to shipping.** Our company stands behind the quality materials, components and workmanship of the Merlin VGA video magnifier.

For these reasons we are pleased to offer two warranty options. These options are available when the purchaser completes the warranty registration form.

Three-year warranty for the Merlin VGA base

This option can be selected on the warranty registration form by the agencies, states, groups, multi-users, product demonstration sites and the individual purchasers. Enhanced Vision warrants Merlin VGA base against defects in materials and workmanship under normal use for three years from the date of original purchase. Enhanced Vision, at its option, will at no charge either repair the product or replace it during the warranty period.

Lifetime warranty for the Merlin VGA base at a one-time cost of \$299.00.

This option can be selected on the warranty registration form and it is only available to the individual purchaser. Enhanced Vision warrants Merlin VGA base against defects in materials and workmanship under normal use for the lifetime of the individual purchaser. Enhanced Vision, at its option, will at no charge either repair the product or replace it during the warranty period.

The lifetime warranty is non-transferable and only valid for an individual purchaser. This warranty is not available for International Customers.

On the warranty card the purchaser will be asked to select between the three-year (no-cost) or lifetime warranty for a one-time fee of \$299.00. If lifetime coverage is elected, a \$299.00 check made payable to "Enhanced Vision" will need to be included with the warranty form.

The lifetime product warranty is only valid if the product is registered within 30 days of purchase and a \$299.00 check made payable to "Enhanced Vision" is included with the warranty registration form. If this form has not been provided, please request one from the local Enhanced Vision representative or call us at (800) 440-9476.

Restrictions

All product purchases need to be registered with Enhanced Vision. Registering the product confirms the right to maximum protection and service of the product warranty. If no selection is made on the warranty form or payment is missing, the individual purchaser will default to the three-year warranty.

The lifetime warranty is non-transferable and does not apply to groups, multi-users, companies or agencies. It has been designed for end-consumer / individual purchaser. Agencies, states, group homes, classrooms and demo site units will receive Enhanced Vision standard 3-year product warranty detailed above. Enhanced Vision also reserves the right to repair or replace any purchasers Merlin VGA with a similar or better product.

Warranty Statement

In the case that the end-consumer / original individual resells the Merlin VGA within a 3 year time period from original purchase date and the product is resold by a recognized manufacturer representative, the Merlin VGA warranty will default to the balance of what remains of the standard 3 year warranty (from original purchase date). The lifetime warranty would be void.

Consider the example below:

After one year Mary Smith's vision deteriorates past using the Merlin VGA to assist her. She resells the Merlin VGA to a recognized Enhanced Vision manufacturer representative, and he/she then sells the Merlin VGA to Anne Clark. Anne Clark will receive a four-year warranty with the Merlin VGA (the amount left on a standard three year warranty).

Anne will need to complete and submit a warranty registration form, marking the box that she is a "converted original owner" and provide the serial number for the product.

The 3-year and optional Lifetime product warranty does not cover the following:

1. Defects or damage from misuse, use in other than suggested manner, accident or neglect.
2. Defects or damage from improper operation, maintenance, installation, adjustment, alterations or modification.
3. Defects from spills of food or liquid.
4. All plastic surfaces and all other externally exposed parts that are scratched or damaged due to normal use.
5. Damage due to operating Merlin VGA with equipment not supplied by Enhanced Vision.
6. Light bulbs that are not covered under warranty.

THERE ARE NO OTHER EXPRESSED OR IMPLIED WARRANTIES ACCOMPANYING THIS PRODUCT.

Service Process

We don't expect that you will need to have repairs on the Merlin VGA but if you do, you will need to send the unit to our warranty service center.

If warranty service is required, please call Enhanced Vision between 8 a.m. – 5 p.m. PST for a return authorization number. Warranty repairs are not accepted without a return authorization number.

You may wish to keep your original packaging in case warranty service is required

Proof of purchase must be provided, bearing date of purchase, in order to receive warranty service

Purchaser is responsible and shall pay for shipment of products to Enhanced Vision for repair.

Enhanced Vision is responsible and shall pay for return shipment of repaired products back to purchaser. Enhanced Vision reserves the right to use reconditioned and/or serviceable used parts for warranty repairs.

Anyone using this product knowingly assumes certain risks, due to the need to utilize vision enhancement devices.

Thank you for making Enhanced Vision and Merlin VGA your choice for better vision.

Notes

The Merlin VGA video magnifier is designed to enlarge and enhance images to improve the lives of people who have low vision. The extent to which Merlin VGA can improve the ability to see images and text depends on a person's individual condition and level of visual impairment. Merlin VGA is not designed to restore vision, correct eye conditions or halt degenerative vision loss. An eyecare professional can best determine the suitability of Merlin VGA for a patient's specific condition. For more information, please visit www.enhancedvision.com.

**enhanced
vision**

www.enhancedvision.com

(800) 440-9476 (US Office, toll free)

(714)465-3400 (US Office)

+44 (0) 9442317 (UK Office)

Part # MAN-1100-00-x1
August, 2009

Copyright © 2009 by Enhanced Vision. All rights reserved.
Product specifications subject to change without prior notice.