

Transformer HD iOS Application Patch Notes

New Features & Updates

- Added Support for iOS 12.2 and higher (TRHDE-59)

Bug Fixes & Improvements

- Fixed an issue that caused an application crash when attempting to save an image or video on older iOS Revisions (TRHDE-49)
- Improved OCR and CCTV appearance and scaling for camera preview screens and various device screen sizes (TRHDE-47, 101)
- Added, Fixed or Updated Translations for all languages (TRHDE-133)

Additional Notes

- This release updates the Transformer HD Apple App to be fully compatible with iOS version 12.2 and higher. It is highly recommended that users update to the latest operating system for the best experience. As a result iOS spec is being updated to iOS 12.2.

Known Issues

- OCR Result is poor when loaded from a saved image instead of a live capture (TRHDE-142)
- Camera preview window can be misshapen if user freezes an image and rotates their tablet while in OCR mode (TRHDE-141)
- User must manually exit sub menu functions and cannot simply tap the preview screen to return to Camera Preview (TRHDE-99)
- Users cannot pan around a frozen or saved image (TRHDE-37)
- Outdated iOS Versions may have incorrect vocalizer (TRHDE-48)
- Black and White Color mode shows up as Full Color in CCTV mode (TRHDE-157)
- OCR can sometimes crash and require user to reset Transformer HD Unit (TRHDE-50)
- iOS currently cannot save OCR Text Scans (TRHD-755)
- iOS cannot save or load from TTS Window (TRHD-456)