


Quick Start

1. After removing the Amigo HD from its package plug in the power supply.
2. Press the green power button.
3. Open the stand to the first position.

We recommend you charge the Amigo HD for 2-3 hours prior to first use.

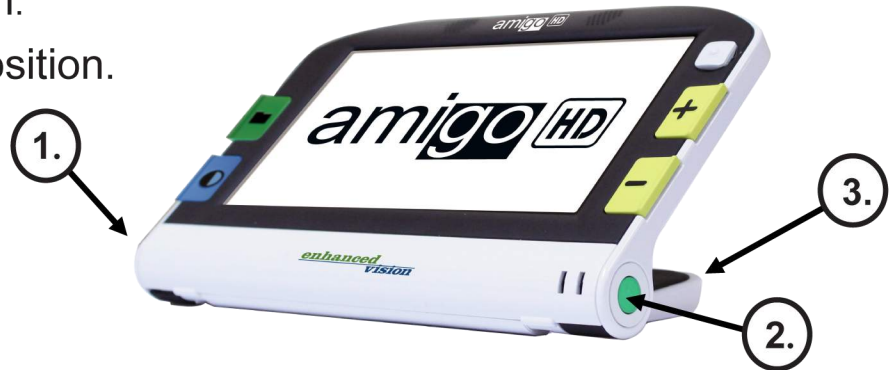

4. Place the Amigo HD on the desired reading material.

5. Press the Blue "Mode" button to select the best picture quality for your vision condition. Keep pressing to cycle through all the choices.

6. Press the Yellow Plus or Minus button to increase or decrease the magnification.

7. If needed adjust brightness using the white joystick by pressing left/right.



8. Your Amigo HD will be set for Auto Focus (AF). If it does not focus, locate and move the MF/AF button to the AF position.

Start reading and enjoying your new Amigo HD.
See the User Manual for more advanced features.