





User Manual

Congratulations on your purchase of the *Merlin elite* HD Magnifier with OCR from Enhanced Vision. The *Merlin elite* is a full-color High-Definition (HD), auto-focus video magnifier system designed to enhance the lifestyle of the visually impaired. With the LCD Monitor's adjustable arm, *Merlin elite* can provide the user with the ultimate flexibility in the position of the screen making reading easier than ever.

The *Merlin elite* is a fully integrated solution, offering the advantage of a built-in HD LCD screen and OCR. *Merlin elite* enables individuals with low vision to read, write, view photos, enjoy crafts and hobbies, and more – in color, black-and-white, or enhanced high-contrast positive or negative viewing modes. *Merlin elite* also includes 3 different preset viewing combinations for both text and background to provide improved viewing when reading. An additional 28 text and background combinations are available and can be programmed. With easy-to-use buttons, you can enlarge or reduce the image in seconds, change viewing modes for easier visibility.

Prior to using this system, please carefully read and follow the instructions provided within this booklet. Your *Merlin elite* is ready to use right out of the box. By reviewing all *Merlin elite* features and controls before use and following the directions carefully, you'll be able to enjoy all the great benefits from your purchase.

Enhanced Vision products are designed to give you the highest quality image, convenient and easy-to-use features and years of reliable service. Thank you for choosing the Enhanced Vision *Merlin elite*.

FCC COMPLIANCE NOTICE

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation. This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- ➤ Consult the dealer or an experienced radio/TV technician for help.

Unauthorized changes or modification to this system can void the users' authority to operate this equipment.

If your Low Vision device has an option to connect to a TV, it must be used with shielded interface cables in order to meet FCC emissions limits for Class B digital devices. It is the responsibility of the user to use the shielded interface cables supplied with the equipment.

Safety and Care 5-9	
Merlin elite at a Glance	
Setting Up Merlin elite	
Control Console	
Button Definitions / Turning Lights On/Off 15-16	
Using the X-Y Table	
Tips for Reading and Writing	
Changing Magnification	
Changing Viewing Modes 20-22	2
Adjusting Brightness & White Balance	
Using and Adjusting Markers24-25	
Volume Adjustment & VGA/HDMI Toggle	
OCR/Text-To-Speech Mode	
Saving, Recalling, Downloading & Deleting Images 31-32	
Screen Saver	
Language Settings/Product Info	
Troubleshooting	
Merlin elite Specifications	
Warranty Statement	
Other Enhanced Vision Products	2

SAFETY AND CARE

For your safety and to obtain the most benefit from the *Merlin elite*, please read these safety precautions very carefully.

The *Merlin elite* is designed to enlarge and enhance images and read print aloud in order to improve the quality of life for people who are visually impaired. The extent to which this product can improve the ability to see images depends on a person's individual condition and level of visual impairment.

The *Merlin elite* is not designed to restore vision, correct eye conditions or halt degenerative vision loss. An eye care professional can determine the usefulness and suitability of the *Merlin elite* for a patient's individual condition.

Precautions for Use

- > Follow all warnings, cautions, and instructions marked on the product.
- ➤ Use the *Merlin elite* on a stable, level surface such as a solid desk or table, according to its intended design specifications.
- ➤ Avoid placing the *Merlin elite* on an unstable table, cart or stand. It may fall causing serious damage to the unit or bodily injury to the user.
- ➤ Use the *Merlin elite* indoors only it should NOT be exposed to extreme or rapid changes in temperature or humidity. Do not place the *Merlin elite* near a radiator, heater or areas exposed to long hours of direct sunlight. These conditions may cause damage to the unit's electronics or optics.
- ➤ At least once per week, clean the *Merlin elite* with a soft dry cloth or an electrostatic dusting wipe. DO NOT use liquid cleaners on its plastic surfaces. No additional methods of sterilization or cleaning are recommended nor required.
- Do not use the Merlin elite near flammable materials or liquids.

Additional Electrical Safety Precautions

- ➤ The power supply cord is a disconnect device for electrical equipment. The socket-outlet should be installed near the equipment and be easily accessible.
- ➤ This product is equipped with a 3-wire grounding type plug as a safety feature. This plug has a third (grounding) pin and can only be plugged into a grounding-type power outlet. If necessary, contact an electrician to replace your power outlet with a grounding-type power outlet.
- ➤ Do not use the *Merlin elite* in, near or around water. The *Merlin elite* is not designed to offer protection against contact with liquids of any kind.
- ➤ Use only the power supply provided with your *Merlin elite*. Using unauthorized power supplies may cause serious damage to *Merlin elite*'s electronic components.
- Using any other power supply than the one provided by Enhanced Vision for the Merlin elite will void the warranty.
- Always place the power cord away from traffic areas to avoid damage of power supply.
- Prevent the risk of electrical shock by NOT attempting to disassemble the unit. Please refer all service to an authorized repair facility.
- ➤ To avoid personal injury or damage to the *Merlin elite*, do not make connections other than those shown in the photo below.





Additional Mechanical Safety Precautions

Your *Merlin elite* contains moving parts. Please use caution when moving the X-Y table, adjusting the LCD monitor arm, or when transporting the *Merlin elite* to avoid possible injury to hands and fingers.



To Move Merlin elite to Another Location

- ➤ Use caution if you are repositioning the *Merlin elite* a short distance across a desk or table. Make sure you do not pull on any power cords or connector cables.
- Disconnect and secure all power and video cables and lock the X-Y table before moving the unit.
- ➤ Carry the unit only by the two recessed handles located on either side of its base.

Do not carry it by the LCD monitor arm.

Caring for Merlin elite

- ➤ Always turn the power OFF (by holding the Power button for 5 seconds) on the *Merlin elite* when the system is not in use.
 - Pressing the Power button on the Front Panel, right side, of the Merlin elite for less than 5 seconds will put the device in Standby. Pressing the Power button again will wake the Merlin elite up from Standby.
- ➤ Always turn the power OFF (by holding the Power button for 5 seconds) before cleaning the Merlin-elite.
- ➤ Clean the *Merlin elite* with a soft dry cloth or an electrostatic dusting wipe. DO NOT use liquid cleaners on its plastic surfaces.
- ➤ The protective glass covering the camera can be cleaned as needed with the lens cloth provided within the packaging of the *Merlin elite*.

Susceptibility to Interference:

There may be temporary degradation of the image when the *Merlin elite* is subjected to a strong radio frequency field, electro-static discharge or transient electrical noise. An electro-static discharge (caused by static electricity) may cause the screen to go blank. If this happens, disconnect the AC power from the electrical outlet, wait 30 seconds, and connect the AC power again.

Compliance Warnings:

- ➤ Electromagnetic compatibility: The use of accessories and cables other than those specified by Enhanced Vision may result in increased emissions or decreased immunity of the *Merlin elite*.
- ➤ Placement of equipment: The *Merlin elite* should not be used adjacent to or stacked with other equipment.

Product enhancements and revisions are an integral part of what we do here at Enhanced Vision. Please visit – http://www.enhancedvision.com/low-vision-product-line.html and select your Enhanced Vision Product. You will find the most up-to-date version of the User Manual at the bottom of the page.

MERLIN elite AT A GLANCE

Take a few moments to familiarize yourself with the components of your *Merlin elite*:

- 1. 24" LCD Screen: displays high quality image
- 2. Power button
- 3. Zoom Wheel (Size/Magnification dial) 'FIND' key in center
- 4. Viewing Mode button
- 5. LCD Monitor Arm
- 6. X-Y table
- 7. X-Y table locking lever
- 8. Recessed handles for carrying *Merlin elite*
- 9. *Merlin elite* 19V power supply
- 10. Merlin elite Power Cord (Attaches to 19v Power Supply)(power cord style may vary based upon country)
- 11. 3.5mm jack for Headphones

Accessories included:

- 12. Control Console: allows the user to control the *Merlin elite* functions
 - *Must be plugged in to use and have access to the menu items for OCR/Text-To-Speech
- 13. Lens Cloth: permits cleaning of the LCD display and lenses



SETTING UP THE MERLIN elite



Figure 1







Figure 4

- 1. Find a steady and level location, such as a table or desk, where you will be comfortable using *Merlin elite*. Carry the *Merlin elite* only by the two recessed handles on the side of the base for a secure grip. Do not lift the unit by the LCD monitor arm. Place it on the table or desk (see Figure 1).
- 2. Connect the Control Console cord (See Page 10 Item 12) to the USB port at the back, left side (facing the back), of the *Merlin elite* base (See Figure 2).
- 3. Connect the power cord to the power supply (See Figure 3). Connect the power cord to an available electrical outlet (See Figure 4). Connect the other end of the power supply to the *Merlin elite* where it is labeled "19VDC" (See Figure 2).

To offer additional protection during an electrical storm, a surge protector may also be used and is recommended.

If the lights on the Power, Zoom Wheel and Mode buttons do not illuminate, or if an image does not appear on the HD LCD monitor, please check the power supply connections (review steps 2-3 on page 12 for details).


Pressing the Power button on the Front Panel, right side, of the *Merlin elite* for less than 5 seconds will put the device in Standby. Pressing the Power button again will wake the *Merlin elite* up from Standby.

To turn the *Merlin elite* off, press and hold the Power Button for more than 5 seconds or until "*Merlin elite* HD/OCR" appears on the screen.

Button Combinations

Throughout the User Manual there will be 'button combinations' (e.g. Press the "[Find] and [Mode]" buttons). This indicates that the buttons should be pressed simultaneously. Control buttons, levers, etc. are enclosed in [].

Merlin elite Control Console

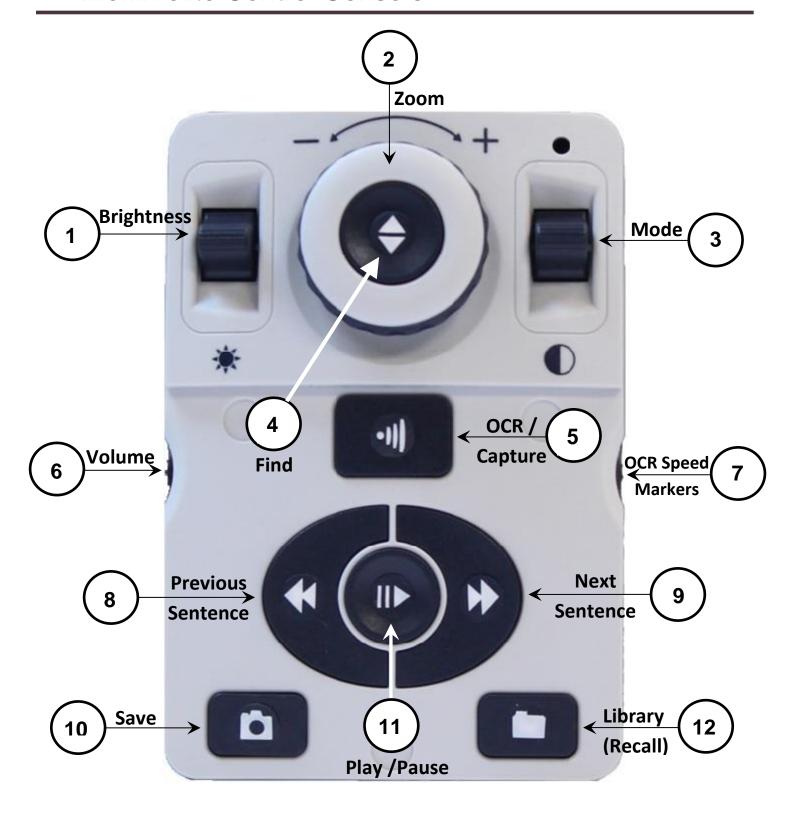

NOTE: When the lower cover is placed on the Control Console, OCR related menu options are de-activated in the *Merlin elite* Menus

See the following Page for Console Controls Detail

BUTTON DEFINITIONS

- 1. Brightness When in desktop/live mode, press Up/Down to increase/decrease brightness
 - When in OCR mode, press [Brightness] Up/Down to move to the Next/Previous word. 'Press and hold' [Brightness] Up/Down to advance 2 words per second. **Note**: Word navigation is only available when OCR is paused.
- Zoom Wheel Turn clockwise/counter-clockwise to increase/decrease the magnification level.
 In the menu or [Recall] mode, turn clockwise/counter-clockwise to advance through the menu
 items or saved files
- Mode Press to advance through viewing modes. 'Press and hold' for 2 seconds to return to 'Normal' color from any 'Enhanced' color mode. Press [Mode Up] for 2 seconds for the color mode menu
- 4. Find When used above Minimum Zoom, a box outline will indicate what will be displayed at that zoom level to assist with document positioning.
 - When used in combination with the [Mode] button, this allows one to enter the Main Menu (press "[Find] and [Mode Up]").
 - When in the menu, press [Find] to select a menu item. To exit menu, press [Mode Up].
- 5. OCR/Capture Press to capture an image for OCR (starts Zoning Preview if enabled in menu). Press again to exit out of OCR and back to desktop/live mode.
- 6. Volume Rotate to increase/decrease volume in desktop/live mode or OCR mode
- OCR Speed / Markers When in OCR mode, rotate clockwise/counter-clockwise to increase/decrease OCR Playback speed (Speed 1-10 only during OCR Playback). Pressing this wheel will reset the OCR Playback speed to 5.
 - When in desktop/live mode, rotate clockwise/counter-clockwise to cycle through enabled markers (Column, Line & Window). Pressing this wheel will remove visible markers.
- 8. Previous Sentence (In OCR Mode) Press to move to the previous sentence. 'Press and hold' for more than 2 seconds to move to a previous sentence, two sentences per second.
 - When in [Recall] jumps to top of current list or jumps to previous bottom of list
- 9. Next Sentence (In OCR Mode) Press to move to the next sentence. 'Press and hold' for more than 2 seconds to move to the next sentence, two sentences per second.
 - When in Recall Jumps to bottom of current list or jumps to top of next list
- 10. Save (desktop/live mode) 'Press and hold' for 2 seconds or more to save the current image. In OCR mode, 'Pressing and hold' for 2 seconds during OCR Playback to save the current OCR
- 11. Pause/Play Toggle operation: Used to pause, or resume play, during OCR Playback
- 12. Library (Recall) Enters the Image/OCR File recall menu (Press for 2 seconds)

Turning Lights On/Off (toggle)

To turn the *Merlin elite* lights On/Off:

Press and hold the [Marker Wheel] for more than 5 seconds

***Note: The *Merlin elite* lights will automatically turn on when it is first powered on ***

NOTE: If your *Merlin elite* is plugged into a power strip/surge protector, make sure the power switch is turned on before attempting to turn on *Merlin elite*. If the power strip/surge protector comes with a circuit breaker, make sure the circuit breaker has been reset before attempting to turn on *Merlin elite*.

USING X-Y TABLE

Locking the X-Y Table

To lock the X-Y table, pull the lever all the way to the front position until it locks (A). Unlock the X-Y table by pushing the lever to the back position (B).

You can create X-Y table 'drag' by positioning the locking lever between 'unlocked' (all the way back) and 'locked' (all the way forward). This will allow for more controlled vertical/horizontal movement of the X-Y table.





Reading With the X-Y Table

- 1. With the X-Y table unlocked, place a book, newspaper, etc., on the table.
- 2. Use the magnification dial to enlarge the image to a comfortable viewing size.
- 3. Glide the X-Y table to the left, which will make the words flow across the screen for reading.
- 4. When you have reached the end of a line, glide the table back to the beginning of the line.
- 5. Move the table away from you slightly to display the start of the next line.
- 6. Repeat steps 4, 5 and 6 to continue reading.

TIPS FOR READING AND WRITING

Reading with Merlin elite

- ➤ New users of video magnifier systems have a tendency to overmagnify their reading material. Make the type large enough to see it comfortably, but not so large that it becomes tedious to scan across a page.
- ➤ Use the X-Y table to move reading materials under the camera, rather than moving the material manually, except small items such as a checkbook or a business card.
- ➤ New users should use two hands when moving the X-Y table for reading, until they become comfortable using one hand.

Writing with Merlin elite

- ➤ Beginners should start at smaller magnification sizes, and then increase magnification as they become more comfortable using *Merlin elite* for writing.
- The smaller the magnification size the bigger you tend to write; the higher the magnification size the smaller you'll write.
- ➤ Use a pen that makes a bold line, such as a heavy point pen, a felt-tip marker or a fine point marker, for easier visibility.
- ➤ It might be helpful to use lined paper to more easily find and keep your place.
- ➤ It might be easier to keep your place if you write a few words at a time, then move the X-Y table and write again, rather than moving continually as you write.
- Write in either color, black-and-white or positive viewing modes.

CHANGING MAGNIFICATION

The *Merlin elite* offers a large range of magnification capabilities. The following directions can be used with the *Merlin elite* Front Panel Controls (See Figure 1) or Control Console (See Figure 2).

- 2. The Zoom Level will appear in the upper right corner of your monitor when you are increasing or decreasing magnification. "Max Zoom," will appear at the maximum magnification and "Min Zoom," will appear at the minimum magnification.

NOTE: The extent of the magnification is dependent upon the distance of the Camera to the object.



Figure 1



Figure 2

Working Distance

You can increase the 'working distance' (distance from source material to camera lens) of the *Merlin elite* by enabling this setting. This feature is best utilized when you have thicker 'source material' (e.g. a thick book) on the X-Y Table

Note: You MAY notice a slight reduction in image quality (above Magnification level 8) when this setting is enabled

CHANGING VIEWING MODES

The *Merlin elite* is designed to provide optimum viewing for people with certain visual conditions by offering a range of choices in both text and background colors.

Press the [Mode] button located on the Front Panel, or Control Console, to toggle between: Full Color viewing, Black and White, Enhanced Positive, Enhanced Negative, and up to 5 of your favorite color select modes. Each push will allow you to cycle through all seven viewing modes. The 5 favorite color select modes can be modified to any of the 28 color select options by following the instructions provided on page 22.

VIEWING MODES

Full Color Image (default)

This mode gives a true color representation of the image being magnified.

Black-and-White Image

This mode gives a black and white representation of the image being magnified.

Enhanced-Positive Image

This mode provides a white background with black text. It increases the contrast to offer crisp text images for reading.

Crisp, Clear Text Images!

Enhanced Positive

Enhanced-Negative Image

This mode provides a black background with white text. It increases the contrast to offer crisp text images for reading.

Crisp, Clear Text Images! Enhanced Negative

Color Select 1

In factory default this mode provides a black background with green text. This is one of the most commonly used color combinations. To change this mode to any of the 28 available follow the instructions on page 22.

Color Select 2

In factory default this mode provides a black background with yellow text. To change this mode to any of the 28 available follow the instructions on page 22.

Color Select 3

In factory default this mode provides a yellow background with blue text. To change this mode to any of the 28 available follow the instructions on page 22.

Color Select 4

In factory default this mode is turned off. Follow the instructions on page 22 to turn this color mode on.

Color Select 5

In factory default this mode is turned off. Follow the instructions on page 22 to turn this color mode on.

COLOR SELECT MODE MENU

'Press and hold' the [Mode] button for 3-5 seconds on the Front Panel, or Control Console, to bring up the Color Select Menu. Scroll through the available Color Select Modes by turning the Zoom Wheel.



Press the [Find] button on the Front Panel (Center of Zoom Wheel), or Control Console, to select the highlighted Color Mode.

Changing Favorite Color Modes

The *Merlin elite* provides 28 selectable Color Select Modes and allows selecting up to 5 favorites for use. The following directions can be used to change the favorite color modes on your *Merlin elite*.

- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- **Brightness EXIT BLACK & WHITE OCR Settings** GREEN/BLACK **Markers Setting** YELLOW/BLACK **Zoning Preview** RED/BLACK Color Mode Selection BLUE/BLACK System Configuration PINK/BLACK Primary Language ORANGE/BLACK Working Distance BLACK/GREEN Product Information BLACK/YELLOW BLACK RED
- 2. Turn the [Zoom Wheel] clockwise until "Color Mode Selection" is highlighted
- 3. Press the [Find] button to select the "Color Mode Selection" sub menu.
- 4. Turn the [Zoom Wheel] until the desired Color Select Mode is highlighted.
- 5. Press the [Find] button to select or deselect the highlighted color mode.
- Once 5 Color Select Modes have been selected no other color modes can be selected. To choose another Color Select Mode, deselect one of the 5 Color Modes previously selected.

Adjusting the Brightness

- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- 2. "Brightness" should already be highlighted. Press the [Find] button to select the brightness adjustment option.
- 3. Turn the [Zoom Wheel] clockwise to brighten the display or counterclockwise to dim the display.
- 4. Once the desired brightness is set, press the [Mode] button to exit or wait 10 seconds for the Menu to exit.

Adjusting the White Balance

The color white is often affected by the lighting used in a room. The *Merlin elite* White Balance selection allows you to achieve optimum clarity.

- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- 2. Turn the [Zoom Wheel] clockwise until "System Configuration" is highlighted.
- 3. Press the [Find] button to select the "System Configuration" submenu.
- 4. Choose between Auto, Normal, Warm, or Cold by highlighting the desired White Balance.

"Color: Auto" - Automatically adjusts to the lighting in the room

"Color: Normal" - Casts a balanced tone

"Color: Warm" - Casts a red tone

"Color: Cold" - Casts a blue tone

- 5. Press the [Find] button to select the desired setting.
- 6. Press the [Mode] Dutton to exit the menu.

USING LINE MARKERS, WINDOWING & COLUMN MARKERS

Line Markers and Windowing

Line Markers can be used as a reading guide by providing horizontal lines across the screen.

Windowing can be used as a reading guide by only displaying a portion of the screen.

Turning Markers On/Off:

Rotating the [Marker Wheel] on the Control Console will allow you to toggle through the Marker Options
*Column Marker is only available when Control Console is plugged in.
Off • Column Marker • Line Marker • Windowing


Adjusting Line Markers and Windowing (Column Marker Cannot be adjusted):

To adjust the position of the Marker:

- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- 2. Turn the [Zoom Wheel] until "Marker Position" is selected. Then press the [Find] button to select.
- 3. When this option is selected it will be highlighted RED. When this occurs, you can turn the [Zoom Wheel] clockwise / counterclockwise to move the Line Markers or Visible area up/down.

Control Console Shortcut:

Press and hold the "[Marker Wheel] and press [Brightness Up/Down]"

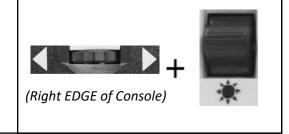


To adjust the size between the Line Markers or size of the Windowing:

- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- 2. Turn the [Zoom Wheel] until "Marker Size" is selected. Then press the [Find] button to select.
- 3. When this option is selected it will be highlighted RED. When this occurs, you can turn the [Zoom Wheel] clockwise/ counterclockwise to increase/decrease the size between the line markers or the size of the visible window.

Control Console Shortcut:

Press and hold the [Marker Wheel] and turn the [Zoom Wheel] clockwise/counter-clockwise

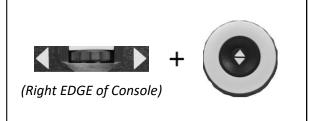

SYSTEM CONFIGURATION SETTINGS-BEEP, VOICE PROMPT, CLOCK, SCREEN SAVER, AND SAVE & RECALL

System Configuration

- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- 2. Turn the [Zoom Wheel] clockwise until "System Configuration" is highlighted.
- 3. Press the [Find] button to select the "System Configuration" sub menu. To select an option, turn the

[Zoom Wheel] to highlight the desired option. Press the [Find] button to select/deselect that option. A black box next to that option indicates that it is ENABLED.



EXIT		

VOLUME ADJUSTMENT & VGA/HDMI TOGGLE

Adjusting the Volume of your Merlin elite in CCTV mode

On the *Merlin elite's* Control Console you can turn the [Volume Wheel] (Left EDGE of Console) to increase or decrease the volume.

A message on the top right corner of the screen will display the volume level when adjusting the volume. The Merlin-elite provides up to 10 volume levels.

VGA/HDMI Toggle

At the back of the *Merlin elite* is an HDMI Dongle and VGA port which allows for a computer signal to be attached to the *Merlin elite* display. These ports makes it possible to use the *Merlin elite* display as a computer monitor to save desk space.



- ➤ To toggle from the camera image to the computer channel images press the "[Find] and [Pause/Play]" buttons on the Control Console to toggle through CCTV, VGA and HDMI Video channels.
- ➤ To toggle from the computer image back to the camera image, press the [Find]

 button on the Front Panel, or Control Console, of the Merlin elite.

Auto-config while in PC Input Mode

Pressing [Mode] while in PC Input Mode will automatically adjust the Position of the PC image

- ➤ When using the *Merlin elite* Display as a computer monitor, the controls for the camera (e.g. Zoom, Mode, & Marker) will not work for the computer image.
- ➤ Using the *Merlin elite* HD LCD as a computer monitor is similar to attaching a typical computer monitor to your computer or connecting a secondary display to your laptop. Refer to your computer or laptop user manual on how to connect a display. The *Merlin elite* HD LCD supports resolutions up to 1920 x 1080

OCR/TEXT-TO-SPEECH MODE

Using the OCR

Merlin elite provides users with the ability to scan a document placed underneath the camera and read it aloud. OCR/Text-to-Speech is only available when the Merlin elite Control Console is plugged in.

- ➤ If the document contains multiple columns, place the desired column in the middle of the screen. Using the "Column Marker" will help in locating the column the *Merlin elite* will read aloud (see page 24)
- ➤ Turning the "Deskew" option on will allow for a larger margin of error for a document that is not perfectly straight. (see page 29)
- 1. Place the document under the *Merlin elite* camera as straight as possible.
- 2. Press the [Capture] button on the Control Console.
 - This will take a snapshot of the document and process it for OCR.
- 3. After the *Merlin elite* has completed the processing step it will begin to read the document aloud.
- 4. To exit OCR/Text-to-speech Mode, press the [Capture] button on the *Merlin elite* Control Console.

NOTE: If, during the OCR process, you encounter a "No Text Found" fault – reposition the document before attempting to [Capture] again. If the 'fault' continues you may see the Merlin elite restart itself.




Zoning Preview

When enabled, this feature allows you to see what the OCR function is going

to read when the [Capture] button is pressed on the Control Console. If you need to adjust the source material, you can do so and the OCR function will redisplay an adjusted zoning preview before reading it.

Using Zoning Preview

- 1. While in Desktop Mode, place the document under the Merlin elite camera as straight as possible.____
- 2. Press the [Capture] button on the Merlin elite Control Console. The Merlin elite will say "Position Document". Once you have the document positioned as you desire, and Merlin elite detects no movement, Merlin elite will say "Selecting Text". Merlin elite will then display the 'Zoning Preview'.

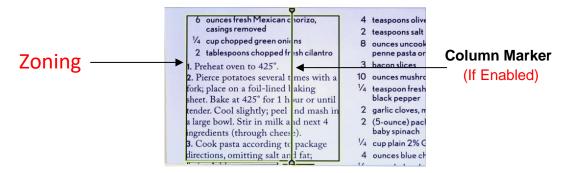

NOTE: It is important to let each of the above steps (Position Document and Selecting Text) complete "prior" to pressing the [Capture] button the 2nd time – pressing it too soon can result in the Zoning Preview being bypassed.

If you are satisfied with the Zoning Preview, press the [Capture] button on the Control Console again. Merlin elite will process the document for OCR Playback.

If you are not satisfied with the 'Zoning Preview', simply reposition the document – Merlin elite will automatically repeat the process and display the new Zoning Preview.

Once you are satisfied with the way Merlin elite is going to 'Zone' your material, press the [Capture] button on the Control Console again. Merlin elite will process the document for OCR Playback.

Adjusting Volume (OCR Playback Mode)

➤ To increase/decrease the volume, rotate the [Volume Wheel] on the Control Console.

Note: CCTV and OCR mode volume levels are independent from each other

Adjusting Reading Speed

➤ To increase/decrease the speed that the OCR will read the document aloud, turn the [OCR Speed/Marker Wheel] (Right EDGE of Console).

Skip to Next or Previous Sentence

➤ To skip to or go back to the previous sentence, press the [Previous Sentence] or [Next Sentence] button on the Control Console. 'Press and hold' for more than 2 seconds to move to a previous sentence, two sentences per second.

Skip to Next or Previous Word

To skip to, or go back to, the next/previous word, press [Brightness Up/Down] to move to the Next/Previous word. 'Press and hold' [Brightness Up/Down] to advance 2 words per second. **Note**: Word navigation is only available when OCR Playback is paused.

Pause/Play

- Press the [Pause/Play] button on the Control Console to pause OCR Playback.

Using Headphones with the Merlin elite

On the left hand side of the *Merlin elite* monitor, there is an available 3.5mm input jack for use of standard headphones.

*Note: Sound from the *Merlin elite* speakers will not be available while the headphone jack is in use.

*Headphones are not included with *Merlin elite*.

OCR Menu Settings

Male Voice/Female Voice

Set the *Merlin elite* to read aloud or prompt in a Male or Female Voice.

Deskew

When on, the *Merlin elite* will automatically adjust a document when not set straight under the camera.

Progress Bar

Enables/Disables the visibility of a progress bar when the *Merlin elite* is processing a document to Text-To-Speech.



Saving and Recalling Images

The *Merlin elite* has the ability to save images from CCTV or OCR modes. This will allow the user to take a snapshot of a document, save it, and view it at a later time. Saving an Image

- 1. Place a document underneath the camera head and set the desired Color Mode and Zoom.
- 2. Press and hold the [Save] button on the *Merlin elite* Control Console for 2 seconds. When the image is saved the *Merlin elite* will display "IMAGE SAVED". If the storage space is full, the *Merlin elite* will display "DEVICE FULL" with a voice prompt (if enabled)

The *Merlin elite* will name that image by the month, day, year, time, and mode it was saved at.

mm.dd.yyyy – hh:mm [OCR/IMG]

*In CCTV mode, if you save the image consecutively within a minute, the previous image saved will be overwritten.

**In OCR mode, if you save the image consecutively within a minute, the *Merlin elite* will display "Already Saved"

Recalling a Saved Image

- 1. Press and hold the [Recall] button on the Merlin elite Control Console for 2 Seconds.
- 2. The *Merlin elite* will enter the Recall Image menu displaying a list of available images.

If the storage space is empty, the message "NO FILES" will be displayed with a voice prompt (if enabled).



- 3. Use the [Zoom Wheel] on the *Merlin elite* Front panel, or Control Console, to navigate through the list of images. When an image file on the list is highlighted, it will display a preview of that image on the right side of the screen.
- 4. Pressing the Find button on the highlighted file will display the image in full screen and begin to read aloud if saved in OCR mode.
 - To exit the Recall Menu back to CCTV mode, press the [Capture] button on the *Merlin elite* Control Console.
 - To exit the full screen display back to CCTV mode, press the [Capture] button on the *Merlin elite* Control Console.

Downloading a Saved File

Recommended using USB hub with a minimum of 2 ports available along with an empty 1GB (minimum) External USB/ Mass Storage Drive

- 1. While in the Recall menu use the [Zoom Wheel] on the Merlin Elite Front Panel or Control Console to navigate to a file.
- 2. Press and Release the [Pause/Play] button on the Merlin Elite Control Console on the desired file.

At this point insert a USB (or Mass Storage Device) to the USB port on the back on the Merlin Elite

- 3. A sub menu will appear next to the selected file allowing you to "CANCEL" "DELETE". or "DOWNLOAD"
- 4. Use the [Zoom Wheel] on the Merlin Elite Front Panel or Control Console to select "DOWNLOAD". (At this stage the storage device is already connected to the Merlin Elite.)
- 5. Press the [Find] button on the Merlin Elite Front Panel or Control Console while "DOWNLOAD" is highlighted to select, and then wait for the "Download Complete" prompt before removing the storage device.

Downloading ALL Saved Files

Recommended using USB hub with a minimum of 2 ports available along with an empty 1GB (minimum) External USB/ Mass Storage Drive

- 1. While in the Recall menu Press and Hold the [Pause/Play] button for about 10 seconds on the Merlin Elite Control Console. A sub menu will appear with the options to "CANCEL", "DELETE ALL", or "DOWNLOAD ALL".
- 2. Insert a USB (or Mass Storage Device) to the USB port on the back on the Merlin Elite
- 3. Use the [Zoom Wheel] on the Merlin Elite Front Panel or Control Console to select "DOWNLOAD ALL". Wait for the "Download Complete" prompt before removing the storage device.

Deleting a Saved Image

- 1. While in Image Recall Menu use the [Zoom Wheel] on the *Merlin elite* Front Panel, or Control Console, to navigate to an image.
- 2. Press the [Pause/Play] button on the *Merlin elite* Control Console on the desired image file.
- 3. A sub menu will appear next to the selected image allowing you to "CANCEL" or "DELETE".
- 4. Use the [Zoom Wheel] on the Front Panel, or Control Console, to select "CANCEL" or "DELETE".
- 5. Pressing the "Find" button on *Merlin elite* Front panel, or Control Console, while "CANCEL" is highlighted will bring you back to the Image Recall menu. Pressing the [Find] button on *Merlin elite* Front Panel, or Control Console, while "DELETE" is high- lighted will delete the selected image. The *Merlin elite* will display "IMAGE DELETED" when the file has been successfully deleted. Press the [Mode] button on the Front Panel, or [Mode Up] on the Control Console, to exit the Image Recall menu.

Merlin Elite Screen Saver

When the Screen Saver feature is turned on, the Merlin elite will display the date and time on the entire screen if there is no activity for 10 minutes. Any movement under the camera or pressing any button on the Merlin elite Front Panel, or Control Console, will revert the Merlin elite back to its previous state. It is possible to manually display the screen saver by pressing the [Pause/Play] and [Save] buttons at the same time.

12:02 PM
Friday
January 06, 2012

Language Setting

The *Merlin elite* supports multiple languages that can be set in the Primary Language menu.

Setting the Primary Language

The primary language that is set will be the main language for the *Merlin elite*. The Menu text and prompts that will be displayed on the *Merlin elite* will be in the language that the user selects in this menu.

Brightness	EXIT	
OCR Settings	1. English (American)	0
Markers Setting	2. Françias (Canadian)	כ
Ivial kers setting	3. Italiano	כ
Zoning Preview	4. Español (Castilian)	
Color Mode Selection	5. Español (Mexican)	ס
System Configuration	6. Deutsch	כ
Primary Language		
Working Distance		
Product Information		

- 1. Press the "[Find] and [Mode]" buttons located on the *Merlin elite* Front Panel, or Control Console, to enter the Menu.
- 2. Turn the [Zoom Wheel] Oclockwise until "Primary Language" is highlighted.
- 3. Press the [Find] button to select the "Primary Language" sub menu. To select an option, turn the [Zoom Wheel] to highlight the desired language. Press the [Find] button to select that option. A black box next to that option indicates that that option is ON.

Setting the OCR Language

The Merlin elite is capable of reading text aloud in multiple languages. By default, the Merlin elite will only read aloud text written in the same language that is set in the Primary Language menu.

By selecting an additional OCR Language, the user can select additional languages, other than the previously set primary language; the *Merlin elite* will be able to read text written in that additional language aloud.

- **Brightness EXIT OCR Settings** Male Voice Female Voice **Markers Setting** Deskew **Zoning Preview** Progress Bar Color Mode Selection 1. English-British System Configuration 2. Français **Primary Language** 3. Italiano Working Distance 4. Español(Castilian) **Product Information** 5. Deutsch 6. Nederlands
- 1. Press the "[Find] and [Mode]" buttons located on the Front Panel, or Control Console, of the *Merlin elite* to enter the Menu.
- 2. Turn the [Zoom Wheel] oclockwise until "OCR Settings" is highlighted.

- 3. Press the [Find] button to enter the "OCR Settings" menu options.
- ➤ To select an additional OCR Language, turn the [Zoom Wheel] button to select highlight the desired language. Press the [Find] button to select that option. A black box next to that option indicates that it is ENABLED. The Primary Language will already be selected and the user will not be able to deselect that option. Multiple languages can be set at one time.

Product Information

The Product Information will display the technical information of the Merlinelite and also allows the user to restore the *Merlin elite* back to its factory settings.

- 1. Press the "[Find] and [Mode]" buttons located on the *Merlin elite* Front Panel, or Control Console, to enter the Menu.
- 2. Turn the [Zoom Wheel] clockwise until "Product Information" is highlighted.
- 3. Press the [Find] button to display the "Product Information" submenu.
- 4. While in this menu you can:
 - a. Check the firmware versions and other software related version numbers.
 - b. Or restore the settings of the *Merlin elite* by rotating the [Zoom Wheel] clockwise to "Factory Restore" then pressing the [Find] button.

** When resetting the *Merlin elite* to its

factory settings, the Male/Female voice and Primary Language preference will remain the same.

^{**} Languages may vary upon region and can change with future updates.

Brightness OCR Settings Markers Setting Zoning Preview Color Mode Selection PRD S/N: <NOT FOUND> Board S/N: <NOT FOUND> S/W R/N: 03.1.0 System Configuration Boot R/N: 02.04 Cam R/N:: 0020-045e-0100 FP R/N: 01-10 **Primary Language** MAC: 00-80-0F-11-70-00 Language Group 1 **Working Distance EXIT Factory Restore Product Information**

TROUBLESHOOTING

The Merlin elite will not power on

- ➤ Push the Power button on the *Merlin elite's* Front Panel. Lights on the Power button, Zoom Wheel and Mode button should illuminate indicating the power is turned on.
- ➤ Make sure the power supply from the *Merlin elite* is securely connected to a working outlet.
- Make sure the two parts of the power supply cord are securely connected.
- ➤ Make sure the power supply cord is completely plugged into the DC 19V IN jack on the back of the viewing stand base.
- ➤ Check the power strip/surge protectors on/off switch and circuit breaker if the *Merlin elite* is plugged into one.

The images seem to be out of focus

- ➤ Clean the protective glass covering the camera with the nonabrasive lens cleaning cloth provided with your *Merlin elite*.
- Clean the monitor screen with a non-abrasive cloth sprayed with a glass cleaning solution.
- ➤ Make sure the target material is steady on the tray. It might take a few moments for the camera to achieve perfect focus after motion has stopped. This is normal.
- ➤ Make sure the LED Lamps near the *Merlin elite* Camera are ON. Making sure there is sufficient ambient lighting will assist in creating a better image.

X-Y Table Does Not Move Smoothly

Make sure the locking lever is positioned completely to the back.

X-Y Table Does Not Stay in Locked Position

Make sure the locking lever is all the way to the front.

The *Merlin elite* is taking a long time to process the document for Text-to-speech.

Disabling the "Deskew" in the menu will increase the processing speed. By disabling "Deskew" in the document must be placed under the *Merlin elite* camera as straight as possible.

Resetting the Merlin elite

If the *Merlin elite* is not responding you can 'reset' it by pressing the "[Find] and [Power]" buttons. Any setting changes you have made will be retained.

Replacement LED lamps

Your *Merlin elite* comes equipped with two 100,000-hour LED lamps designed to provide years of trouble-free service. However, in the unlikely event that the LEDs need replacement, contact your place of purchase

or Enhanced Vision at (800)440-9476 for instructions or assistance on replacing the LEDs.

Firmware Update

To keep your *Merlin elite* up to date visit our software support page at: http://www.enhancedvision.com/support/software.html

If you have any questions or require technical assistance with your product, please call (800) 440-9476,

Monday – Friday, from 8 a.m. to 5 p.m. PST.

MERLIN ELITE SPECIFICATIONS

Product

➤ Dimensions:

24" LCD: 27.75" H x 22.75" W x 20" D (70.5cm H X 57.8cm W x 50.8cm D)

Product Weight: 24" LCD: 40.1 lbs (18.2 Kg)

Display

Type: 24" Standard Diagonal LCD Average lifetime: 40,000 hours

Resolution: 1920 x 1080 (24" model)

Response Rate: 1.5 milliseconds (24" monitor) All monitor specifications subject

to change without prior notice

Magnification range

24" LCD: 2.4x (minimum) - to 73.2x (maximum) Field of View (Reading Mode): 7.1mm (min); 220mm (max)

Viewing Modes

Color, black & white, high contrast Positive/Negative, Digital Color Select (3 modes).

Power Supply

Input voltage/current: 120-240 VAC / 0.3-0.5A (nominal)

Output voltage: 19 VDC Frequency: 50/60 Hz Power usage: 30W

Equipment Classification (IEC 601-1):

Type of Protection: Class 1 Degree of Protection: Type B Mode of Operation: Continuous

Operating temperature range: 0-40°C / 32-104°F

Storage/transport temperature range: 0-60°C / 32-140°F

Warranty: 3 Years

^{*}Product specifications subject to change without notice.

WARRANTY STATEMENT

Enhanced Vision Merlin elite Warranty Policy

Enhanced Vision products are designed to give you the highest quality image, convenient features, ease of use and reliable service. **The Merlin elite** has been inspected and tested prior to shipping. Enhanced Vision warrants the Merlin elite against defects in material and workmanship under normal use for three years from the date of original purchase. Enhanced Vision, at its option, will at no charge either repair the product or replace it during the warranty period. There are no other expressed or implied warranties accompanying this product.

This warranty does not cover: a) Defects or damage from misuse, use in other than normal and customary manner, accident or neglect; b) Defects or damage from improper operation, maintenance, installation, adjustment, alterations or modification; c) Defects or damage from spills of food or liquid; d) All plastic surfaces and all other externally exposed parts that are scratched or damaged due to normal use; e) Damage due to operation of the *Merlin elite* with equipment not supplied by Enhanced Vision.

- ➤ You must provide proof of purchase, bearing the date of purchase, in order to receive warranty service.
- ➤ Anyone using this product does so knowingly assuming certain risks, due to the need to utilize vision enhancement devices.
- ➤ A 2-year extended warranty option is available within 12 months from the date of purchase. Please contact Enhanced Vision for detailed information.

Service Process

In the event that technical support or repair service is needed, please contact your local Enhanced Vision representative or the Enhanced Vision warranty service center.

If warranty service is required, please contact Enhanced Vision at (800)440-9476 Monday through Friday, between 8 a.m. and 5 p.m. Pacific Standard Time (PST) to request a return authorization number. Warranty repairs will not be accepted without a valid return authorization number.

You may wish to keep the original product packaging in case warranty service is required. The manufacturer cannot provide packaging for warranty repairs.

NOTE: Please completely fill out and send in the enclosed warranty

OTHER PRODUCTS FROM ENHANCED VISION



DaVinci

DaVinci is a high performance desktop video magnifier (CCTV), featuring HD, text-to-speech (OCR) and a 3-in-1 camera. With HD you will experience high definition color and contrast giving you a beautiful, crystal clear picture and vibrant colors. Experience the joy of reading with our new text- to-speech feature. DaVinci will read any printed text aloud with the push of a button.

Magnification: 24" LCD: 2X (min.) - to 80X (max.)



Pebble-mini

Pebble-mini's ultra-compact design makes it the ideal hand-held magnifier whether you're home or on the go. Easily read prescriptions, mail, labels, price tags, menus and so much more. 3.0"LCD screen. Freeze frame and image capture. 28 available viewing modes.

Magnification: 2X to 10X



Pebble HD

Pebble HD boasts a new HD camera providing a crisp, clear, colorful, high definition picture. A new ergonomic lightweight compact design makes it the perfect companion whether at home or on the go. Carry Pebble HD in your purse, pocket, or clip it on your belt with the included carrying case.

Magnification: 1.25X – 13.5X



 $C \in$



Transformer

Transformer is the most flexible and portable solution for reading, writing and viewing magnified images at any distance. Compatible with popular magnification software and the latest Windows operating systems as well as Mac OS X. The Transformer's unique design is perfect for work, home or school. Weighing less than 3 pounds. Battery operated up to 4 hours. 28 custom color select modes.

Magnification 2.4X to 30X (based on 17"screen)





Acrobat HD Ultra

ACROBAT HD LCD is a complete system featuring a 3-in-1 Sony HD camera with a 22", 24", or 27" high resolution HD LCD producing maximum picture detail for clear, vivid color display. The adjustable arm, tilting screen, and rotating capability allows the camera to be positioned at any angle for optimal viewing. A built in handle allows for easy transport.

Magnification: 1.8X to 70.9X





Amigo HD

Amigo HD is a truly portable desktop magnifier. Optimal viewing is achieved with a tilting 7 inch viewing screen. Amigo HD provides 1.4x to 14x digital magnification and the lightest weight in its class. Amigo HD has an integrated battery and can be taken anywhere.

Magnification: 1.4x – 14x

(With extended zoom up to 25x)





Merlin Ultra HD

Merlin ultra HD's camera technology produces an even truer, more natural image. Easily read books and magazines, read and write letters, manage your financial records, view color photos, do crossword puzzles, and even pursue a favorite hobby.

Magnification:

20" LCD: 1.9x (min) to 51.2x (max) 22" LCD: 2.1x (min) to 55.9x (max) 24" LCD: 2.3x (min) to 61.2x (max)





DaVinci Pro

DaVinci Pro is a high performance desktop video magnifier (CCTV), featuring a Full HD Sony 1080p 3-in-1 camera and Full Page Text-to-Speech (OCR). With Full HD you will experience high definition color and contrast giving you a beautiful, crystal clear picture and vibrant colors. DaVinci Pro allows you to experience the joy of reading again by converting any full page of printed text to speech.

Magnification: 24"LCD up to 77x

CONTACT INFORMATION

Enhanced Vision manufacturers many different types of products tailored to meet the needs of our low vision customers. If you have questions about any of our products, please don't hesitate to give us a call.

Contact Information:

(US Office) Phone: (714)465-3400 Fax: (714)374-1821

(UK Office) Phone: +44 (0)115 9442317 Fax: +44 (0)115 9440720

(DE Office) Phone: +49 (0)6078 969827-0 Fax: +49 (0)6078 969827-33

www.enhancedvision.com

Technical Support:

Main Toll Free (800) 440-9476 ext. 3

Local Phone: (714) 465-3400 ext. 3

E-mail: evsupport@enhancedvision.com

enhanced vision

The *Merlin elite* is a high performance desktop video magnifier (CCTV) - featuring text-to-speech (OCR). With HD you will experience high definition color and contrast giving you a beautiful, crystal clear picture with vibrant colors. Experience the joy of reading with our new text-to-speech feature. *Merlin elite* will read any printed text aloud with just the push of a button. For more information, please visit www.enhancedvision.com.

www.enhancedvision.com

(800) 440-9476 (US Office, toll free)

(714) 465-3400 (US Office)

+44 (0) 115 9442317 (UK Office)

Tel. + 49 (0) 6078 969827-0 (DE Office)

Fax + 49 (0) 6078 969827-33 (DE Office)



Part # MAN-0505-00 Rev 8 April, 2018

Copyright © 2009 by Enhanced Vision. All rights reserved. Product specifications subject to change without prior notice.