

EV Viewer for Mac OS X Installation Guide

Requirements:

The following are system requirements for downloading and running EV Viewer for Mac OS X:

- 1- Mac OS X version 10.8 (Mountain Lion) or 10.9 (Mavericks)
- 2- USB 2.0 or 3.0
- 3- Transformer VGA3 with Firmware version 3.04 or later

NOTE: If your Transformer is VGA3 but running an older version of Firmware, please update the Firmware using a windows laptop by clicking on this link

<http://www.enhancedvision.com/support/software/transformer-firmware-update.html>

Installation Process:

1. Download the EVViewer-v1.0.zip file to a USB stick.
2. Take the USB to your Mac computer and unzip the file.
3. Move the application (EV Viewer.app) to the desktop or drag and drop it to the application dock on the bottom of the screen.
4. You can alternatively open a new Finder window and drag and drop it in this window or under Applications category.

To run the application:

- 1- Connect the Transformer to the Mac computer via USB cable.
- 2- Double click on the EV Viewer icon to open the application like any other Mac application.
- 3- A new small window will open and by default is set to use the integrated Mac camera, "FaceTime Camera" if you are on a MacBook.
- 4- Wait for few seconds. The Transformer light should go on indicating that the Transformer is live and started communicating with the Mac computer.
- 5- The EV Viewer will automatically switch to the transformer live CCTV camera.
- 6- At this point you should start seeing live image from the Transformer camera.
NOTE: If you don't see live image, that means the Transformer did not sync with the application. Press the reset button on the transformer and wait for few seconds, the transformer light turns on and the live image should display.
- 7- Start using the application menus and shortcut keys. For quick list of all key mappings, see Help Menu.

Special keys:

- Space bar – acts as freeze and unfreeze, toggles between the two
- Esc key – exits out of freeze mode, full-screen mode, basically acts as cancel
- Visit Help menu on the application title bar for further details on key maps or reference the attached document